



Collection Instructions

- The Kit Includes:**
- 1) 2 Cotton Swabs Per Person
 - 2) 1 Brown Collection Envelope Per Person
 - 3) One Larger White Envelope
 - 4) Pre Paid FedEx Envelope For Shipment

Step 1: Swab the inner cheek

- Each collection requires 2 swabs provided in the kit
- Swab the inside of the cheek with medium pressure for twenty (20) seconds
- Repeat with the second swab

Please note that you collecting skin cells, not just saliva. Use enough pressure to ensure you gain enough cells without harming the individual

Step 2: Seal the swabs inside the paper envelopes

- Place the 2 swabs for each individual into the pre-prepared brown envelope which will already have their name documented.
- If any additional information is needed on envelope there will be a notation on the envelope
- Seal each envelope with tape. **Do not lick the envelopes!**

Step 3: Combine and send the DNA samples

- Place the brown envelope for each person collected into the large white envelope and seal with tape
- Place the white envelope in the pre-paid FedEx envelope
- Drop off the FedEx envelope to a FedEx location or a drop box for shipment

By collecting and submitting a DNA sample each party agrees to the following terms and conditions: I do certify that I have the legal right to consent to the enclosed DNA sample (s) tested for the purpose of determining a biological relationship. I understand that legal chain of custody has not been established and therefore the testing is non-legal.

Please Call (833) 304-0041 With Any Questions